

Module	Attribute	Tag	Notes
	> ^(b) Requested Procedure ID	(0040,1001)	Populated with Requested Procedure ID (0040,1001) from Scheduled Procedure Step Sequence (0040,0100) if provided by Modality Worklist.
	^(b) Performed Procedure Step Start Date	(0040,0244)	Date the Performed Procedure Step was started.
	^(b) Performed Procedure Step Start Time	(0040,0245)	Time the Performed Procedure Step was started.
	^(b) Performed Procedure Step ID	(0040,0253)	Populated with Scheduled Procedure Step ID (0040,0009) if provided by Modality Worklist.
	^(b) Performed Procedure Step Description	(0040,0254)	Populated with Scheduled Procedure Step Description (0040,0007) if provided by Modality Worklist.
	^(b) Performed Procedure Protocol Code Sequence	(0040,0260)	Populated with Scheduled Protocol Code Sequence (0040,0008) if provided by Modality Worklist.
	^(b) Comments on the Performed Procedure Step	(0040,0280)	Populated with Comments on the Scheduled Procedure Step (0040,0400) if provided by Modality Worklist.
General Equipment	Manufacturer	(0008,0070)	Set to "Siemens Ultrasound"
	Institution Name	(0008,0080)	X500 System Presets – Organization Name field.
	Software Versions	(0018,1020)	Set to the DICOM Software Version
	Manufacturer's Model Name	(0008,1090)	Set to "Acuson X500"
General Image	Instance Number	(0020,0013)	Image number in study (1 – n)
	Patient Orientation	(0020,0020)	Always sent as 0 length attribute
Image Pixel	Samples per Pixel	(0028,0002)	Set to 1 for MONOCHROME2 images, 3 for RGB images.
	Photometric Interpretation	(0028,0004)	Set to "MONOCHROME2" or "RGB"
	Planar Configuration	(0028,0006)	Color-by-pixel. Set to 0 for RGB images, not provided for MONOCHROME2 images.
	Rows	(0028,0010)	Set to 480 for NTSC; 547 for PAL. For post-processed images and screen captures, this value may be up to 600.