

MAY 15 2001

**510(k) SUMMARY OF SAFETY AND EFFECTIVENESS  
For Emit® II Plus Phencyclidine Assay**

**1. Manufacturer and Contact Information:**

Manufacturer: Syva Company – Dade Behring Inc.  
20400 Mariani Ave.  
Cupertino, CA 95014

Contact Information: Susan L. Collins  
Regulatory Affairs  
Syva Company – Dade Behring, Inc.  
3403 Yerba Buena Road  
San Jose, CA 95135  
Tel: 408 – 239 - 4412  
Fax: 408 – 239 - 2348

**2. Date Summary Prepared:**  
April 13, 2001

**3. Device Trade Name:**  
Emit® II Plus Phencyclidine Assay

**4. Common Name:**  
Enzyme Immunoassay, Phencyclidine

**5. Device Classification Name:**  
The Clinical Chemistry and Clinical Toxicology Devices Panel have classified "Phencyclidine test system" as Class II, Product Code 91 LCM.

**6. Intended Use:**  
The Emit® II Plus Phencyclidine Assay is a homogeneous enzyme immunoassay intended for use in the qualitative and semi-quantitative analysis of phencyclidine in human urine. These reagents are packaged specifically for use on a variety of Olympus® analyzers.

**7. Device Description:**  
The modified assay is similar to the predicate device with minor differences in the packaging of the product. The modified assay has a smaller fill volume of the reagents into different shaped (wedge) reagent bottles. Both the predicate and modified device reagent bottles are made of the same material (HDPE). The modified reagent bottles incorporate a barcode label with assay specific information and are compatible with the Olympus® AU400/600, AU800/1000 and AU2700 Series Analyzers.

**8. Substantial Equivalence**

The modified device has the same operating principles, design, manufacturing materials, method of manufacture, assay performance characteristics and intended use as the predicate device. In conclusion the modified Emit® II Plus Phencyclidine Assay is substantially equivalent to the predicate Emit® II Plus Phencyclidine Assay.



Food and Drug Administration  
2098 Gaither Road  
Rockville MD 20850

**MAY 15 2001**

Ms. Susan L. Collins  
Regulatory Affairs  
Syva Company-Dade Behring Inc.  
3403 Yerba Buena Road  
San Jose, CA 95135

Re: 510(k) Number: K011161  
Trade/Device Name: Emit<sup>®</sup> II Plus Phencyclidine Assay  
Regulatory Class: Class II  
Product Code: LCM  
Dated: April 13, 2001  
Received: April 16, 2001

Dear Ms. Collins:

We have reviewed your Section 510(k) notification of intent to market the device referenced above and we have determined the device is substantially equivalent to devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (Pre-market Approval), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 895. A substantially equivalent determination assumes compliance with the Good Manufacturing Practice for Medical Devices: General (GMP) regulation (21 CFR Part 820) and that, through periodic GMP inspections, the Food and Drug Administration (FDA) will verify such assumptions. Failure to comply with the GMP regulation may result in regulatory action. In addition, FDA may publish further announcements concerning your device in the Federal Register. Please note: this response to your premarket notification submission does not affect any obligation you might have under sections 531 through 542 of the Act for devices under the Electronic Product Radiation Control provisions, or other Federal laws or regulations.

Page 2

This letter will allow you to begin marketing your device as described in your 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally 809.10 for in vitro diagnostic devices), please contact the Office of Compliance at (301) 594-4588. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its internet address "<http://www.fda.gov/cdrh/dsma/dsmamain.html>".

Sincerely yours,



Steven I. Gutman, M.D., M.B.A.  
Director  
Division of Clinical Laboratory Devices  
Office of Device Evaluation  
Center for Devices and Radiological Health

Enclosure

**Indications for Use Statement**

**510(k) Number (if known):**      **Device Name:** Emit® II Plus Phencyclidine Assay

K01161

**Indications for Use:**

The Emit® II Plus Phencyclidine Assay is a homogeneous enzyme immunoassay with a 25 ng/mL cutoff. The assay is intended for use in the qualitative and semi-quantitative analysis of phencyclidine in human urine. These reagents are packaged specifically for use on a variety of Olympus® analyzers.

The Emit® II Plus Phencyclidine Assay provides only a preliminary analytical test result. A more specific alternative chemical method must be used to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method. Other chemical confirmation methods are available. Clinical consideration and professional judgment should be applied to any drug-of-abuse test result, particularly when preliminary positive results are used.

*Fred Lacy*

\_\_\_\_\_  
(Division Sign-Off)

Division of Clinical Laboratory Devices

510(k) Number K01161

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

\_\_\_\_\_  
Concurrence of CDRH, Office of Device Evaluation (ODE)

Prescription Use  \_\_\_\_\_  
(Per 21 CFR 801.109)

OR

Over-The-Counter Use \_\_\_\_\_

(Optional Format 1-2-96)